

Destock Expiring Meds

Last Modified on 11/18/2019 2:28 pm PST

1. **Log in** using your fingerprint or User ID and Password
 2. Press **Cycle Count** on the touch screen display
 3. Press **Destock Expired Meds**
 4. Enter days to expiration in the **Days to Look Ahead for Possible Expiration** field
 - The date field on the far right will automatically adjust to the calendar date you're looking out to.
 - A negative value in the Days to Expiration field will indicate a medication already past its expiration date.
 5. Select the individual item(s) by place a ✓ in the left-hand column or in the Select All field at the top of the screen to select every item.
 6. Press **Next**
 7. The door/drawer for the first item will **Open**.
 8. Confirm **Quantity on Hand** and **Count Found**.
 9. Enter the **Quantity Removed to Expired Waste**.
 10. Update **Soonest Expiration Date**.
 11. Remove the expired product.
 12. Select **Next** if Destocking multiple expired items
 13. Select **Exit** to log out
-